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EN

WHAT IS PHYSIODOSE PHYSIOLOGICAL SALINE SOLUTION?

Physiodose physiological saline solution is a sterile non-injectable physiological saline solution, without preservatives. Presented in single dose vials, **Physiodose physiological saline solution** is safe, hygienic and easy to use.

Composition

Sodium chloride 0,9 g,
Purified water q.s.p. 100 mL.

WHAT ARE THE INDICATIONS FOR PHYSIODOSE PHYSIOLOGICAL SALINE SOLUTION?

Physiodose physiological saline solution is recommended for babies, children, and adults:

- For nasal hygiene: to irrigate or cleanse dry or stuffed up noses,
- For ocular hygiene: as an eyewash or bath to cleanse the eyes,
- For auricular rinsing,
- For cleansing wounds,
- For inhalation in aerosoltherapy.

HOW IS PHYSIODOSE PHYSIOLOGICAL SALINE SOLUTION USED?

Do not use the same single dose for the different uses. The single-dose container is disposable and for individual use. Detach a vial and open it by twisting off the top.

• FOR NASAL AND OCULAR HYGIENE

Physiodose physiological saline solution may be used daily as a nasal and eye wash 1 to 6 time(s) a day, as needed.

• FOR NASAL HYGIENE

To irrigate or cleanse dry or stuffed up noses.

Directions for Use

Tilt the head back. Delicately insert the tip into one nostril and gently squeeze the vial. Repeat the operation in the other nostril. Lift the head up to allow the mucus to run out and then wipe away the excess. Do not blow your nose violently for 10 minutes after administering.

Precaution for Use

For babies, squeeze drops with a minimum amount of pressure to avoid any risk of contamination of the middle ear.

• FOR OCULAR HYGIENE

As an eyewash or bath to cleanse the eyes.

Directions for Use

Pour a few drops of solution into each eye, taking care not to touch the surface of the eye with the vial. Wipe away excess liquid with sterile gauze.

Precaution for Use

Wait 15 minutes before using any other eye treatment product.

Physiodose

• FOR EAR HYGIENE

By rinsing. For perfect ear hygiene, use **Physiodose physiological saline solution** to flush out solubilised earwax deposits and rinse the external auditory canal after using a surfactant auricular solution.

Directions for Use

Rinse the external auditory canal with **Physiodose physiological saline solution**. Use 1 vial of **Physiodose physiological saline solution** for both ears.

Wipe away the surplus liquid with a gauze pad.

• FOR CLEANSING WOUNDS

Directions for Use

Rinse the wound, using the entire contents of one vial and taking care not to touch the wound with the vial.

Precaution for Use

Wipe away the excess liquid from the unaffected area only.

• FOR INHALATION IN AEROSOL THERAPY

Directions for Use

If the product is used alone, follow the instructions included with the inhalation equipment. If the product is used as a solvent, follow the instructions included with the product for inhalation.

Precautions for Use

Ask your doctor for advice in case of first use or in case of respiratory hypersensitivity, of serious respiratory or cardiovascular pathologies.

HOW SHOULD PHYSIODOSE PHYSIOLOGICAL SALINE SOLUTION BE STORED?

Do not use a single-dose vial that is open or damaged due to the risk of rapid bacterial contamination.

Do not use after the expiry date stated on the single-dose vial. Store below 30°C.

KEEP OUT OF THE REACH AND SIGHT OF CHILDREN.

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