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Instructions for Use – please read attentively

BETADINE® SOOTHING RELIEF Eye Allergy

Read this information carefully before you start using BETADINE® SOOTHING RELIEF Eye Allergy. DO NOT THROW THIS LEAFLET AWAY! It contains important information. If you have any questions about BETADINE® SOOTHING RELIEF Eye Allergy please ask your doctor or pharmacist.

1. What is BETADINE® SOOTHING RELIEF Eye Allergy and what is it used for:
BETADINE® SOOTHING RELIEF Eye Allergy is a product which contains Ectoin®, a cell-protective molecule with inflammation-reducing and membrane-stabilising properties.
BETADINE® SOOTHING RELIEF Eye Allergy is an isotonic solution and can be used to treat and prevent symptoms of allergic conjunctivitis. Itchy, red, or watery eyes can effectively be alleviated.
BETADINE® SOOTHING RELIEF Eye Allergy protects against the harmful influence of allergens and supports the regeneration process of irritated and sensitive conjunctiva.
BETADINE® SOOTHING RELIEF Eye Allergy is suitable for wearers of contact lenses, children and for sensitive eyes.
BETADINE® SOOTHING RELIEF Eye Allergy is free of preservatives.

2. Warnings and precautions

- BETADINE® SOOTHING RELIEF Eye Allergy should not be used in the case of hypersensitivity to Ectoin® or any of the other ingredients of the eye drops.
- If recommended by your doctor/health care professional, BETADINE® SOOTHING RELIEF Eye Allergy may be used after eye/ocular surgery and trauma.
- If allergic reactions occur, the use of BETADINE® SOOTHING RELIEF Eye Allergy must be discontinued immediately.
- To ensure safe hygiene, each eye drop bottle should be used by one person.
- Do not use for more than 6 weeks after opening.

3. What you need to know before using BETADINE® SOOTHING RELIEF Eye Allergy

There are no known interactions between BETADINE® SOOTHING RELIEF Eye Allergy and other eye drops or topical eye ointments.
Wait at least 15 minutes after using BETADINE® SOOTHING RELIEF Eye Allergy before applying any other product or drug into the eye.

4. What should be noted during pregnancy and while breast-feeding

Currently, there are no data available regarding the use of BETADINE® SOOTHING RELIEF Eye Allergy during pregnancy and while breast-feeding.
Consult your doctor or pharmacist for advice, before using this product, if you are:
• pregnant or breast-feeding
• not sure whether you are pregnant or not

5. How to use BETADINE® SOOTHING RELIEF Eye Allergy

Always use BETADINE® SOOTHING RELIEF Eye Allergy according to these instructions for use. Consult your doctor or pharmacist before use if you are not sure.
If not otherwise prescribed by your doctor, please apply 1-2 drops per eye, several times a day. Children 10 years of age and younger should be assisted during the application of BETADINE® SOOTHING RELIEF Eye Allergy.
1. Remove the protective cap of the bottle. Avoid any touching of the exposed tip.
2. Place your index and middle finger on the circular pressure plate on the top of the bottle and the thumb on the bottom of the bottle.
3. Before using BETADINE® SOOTHING RELIEF Eye Allergy for the first time, please turn the tip downwards and press the bottom of the bottle with your thumb about 2-3 times, until the first drops comes out. Do not cut the tip.
4. Bend your head backwards and with the forefinger gently pull your lower eyelid downwards and look up. Hold the bottle as vertical as possible. Avoid direct contact of the bottle tip with your fingers, eyes, or contact lenses.

5. To apply BETADINE® SOOTHING RELIEF Eye Allergy bottle once firmly. Due to the patented fit released.

6. Close your eye slowly to disperse the solution.
7. Repeat the procedure on the other eye.

8. After each use dry the tip and close the bottle.
9. Possible side effects of BETADINE® SOOTHING RELIEF Eye Allergy

10. Date of Revision: April 2016, Rev. 01

Legal Manufacturer:

biotrop AG, Stockumer Str. 28, 58453 Witten,

Manufacturing Site:

Laboratoires Pharmastyle, Z.I. Kraft, 67150 B

10. Date of Revision: April 2016, Rev. 01

®: BETADINE is a Registered Trademark
This product is Povidone-Iodine free

30306 EN-FR-AR NOT123208V02

Explanation of the symbols used on the

CE mark with Notified Body number	Legal Manufacturer
0197	
Lot Number	Use By (Expiry date)
LOT	